

Double Row Rotator Cuff Repair


Twist PEEK screw-in suture anchors with tape
SLiK™ knotless screw-in anchors



SHOULDER


Surgical Technique Guide

Double row rotator cuff repair will be demonstrated with Twist PEEK screw-in suture anchors with tape and **SLiK** knotless screw-in suture anchors.

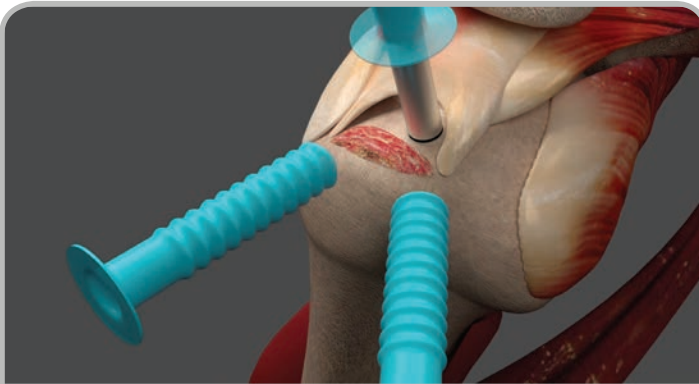


step 1

The rotator cuff footprint is prepared, and pilot holes are created near the articular margin.

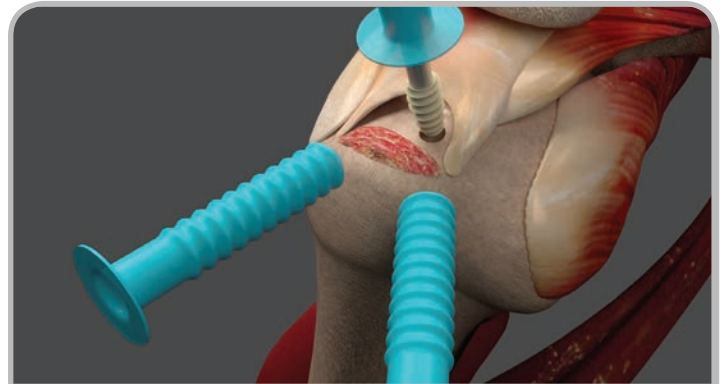


step 2

Insert the Twist, which is pre-loaded with Parcus Braid™ suture tape and #2 Parcus Braid suture.

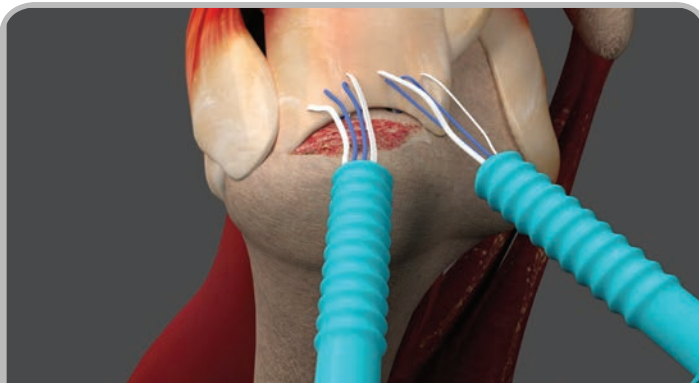


step 3

The suture tape and suture are passed through the rotator cuff in a mattress configuration, using preferred suture passing technique.

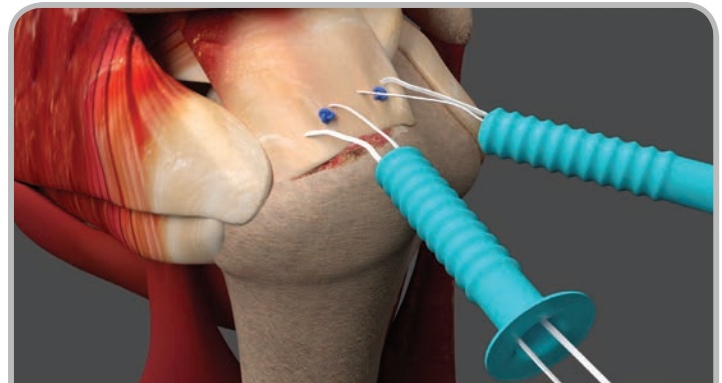


step 4

The paired sutures are tied for primary fixation, using preferred knot tying technique. Tails can be cut or incorporated into lateral row fixation.

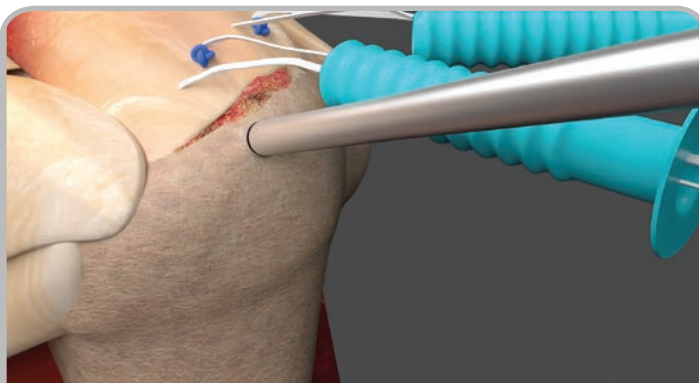


step 5

The lateral row anchor site is identified, and a pilot hole is created using the **SLiK** awl. Repeat this step for each additional anchor.

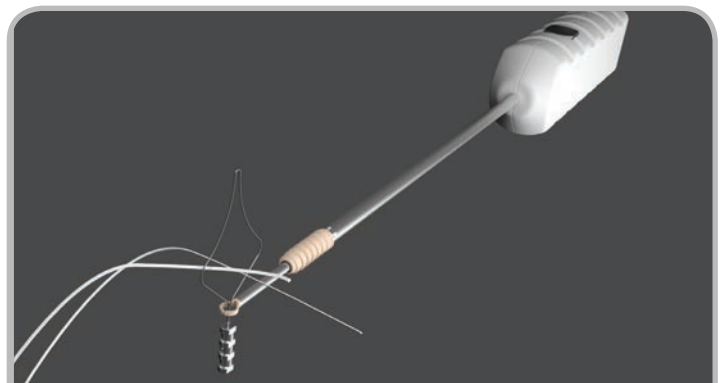


step 6


It is critical that the **SLiK** handle is positioned with the activation button facing upward. The suture tape is loaded top down through the suture guide.

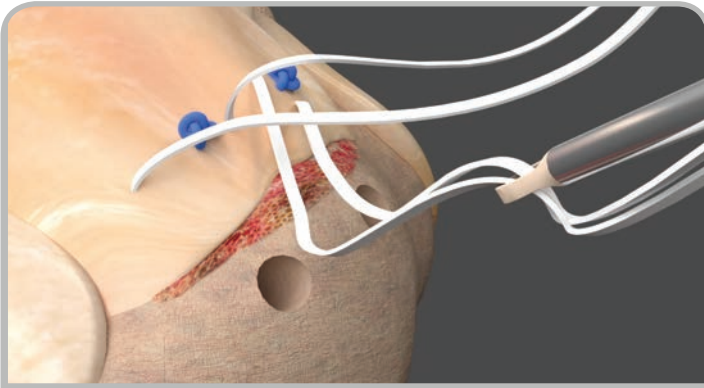


step 7 While maintaining top down orientation, apply light tension on the suture tails. Advance the **SLiK** anchor into the subacromial space through the cannula.

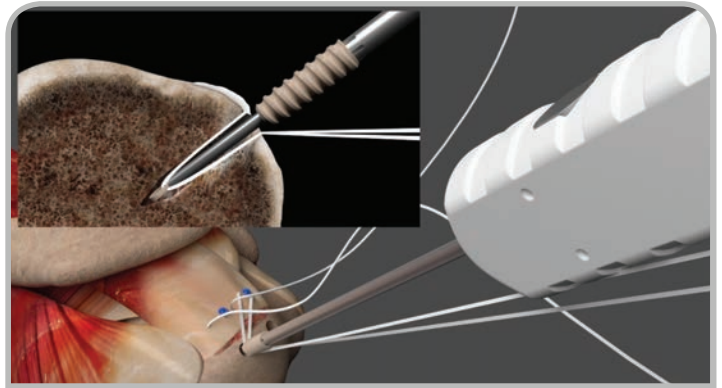


step 8 Keep slight tension on the sutures while inserting the suture guide into the pilot hole. Additional tension can be achieved by individually pulling each suture tail in-line with the handle.

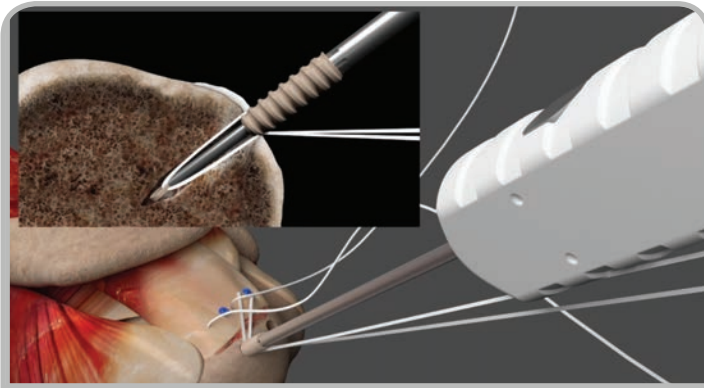

step 9 Once desired tension is set, press the activation button while maintaining downward pressure to release the inner shaft.

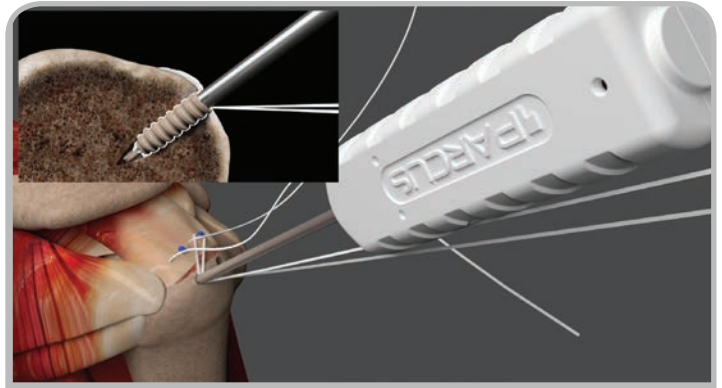

step 10 Turn the handle clockwise to insert the **SLiK** anchor.

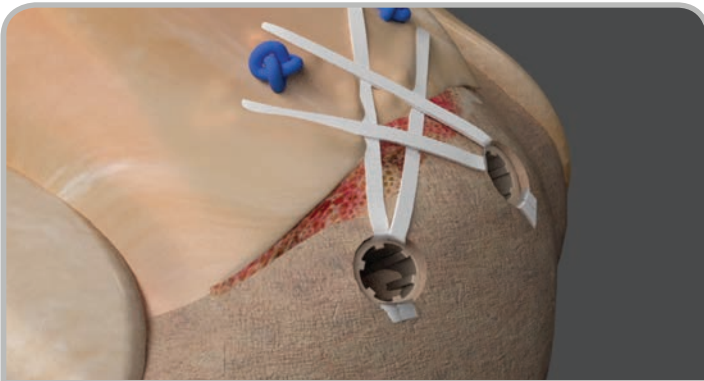

step 11 Once desired depth and final fixation is achieved, pull straight back to remove the handle and release the suture guide. Repeat this step for each additional anchor.

Combining the **SLiK** anchor with Parcus Braid suture tape helps to create an ideal double row rotator cuff repair.